

HOMECATH®

HYDROPHILIC URINARY CATHETERS

Phthalate
Free




Istem
MEDİKAL TIBBİ CİHAZ VE SANAYİ LTD.ŞTİ.



HOME CATH HYDROPHILIC URINARY CATHETER

Homecath Catheter has a superior Hydrophilic coating as well as being manufactured to world standards. Homecath catheters coated with Polyvinylpyrrolidone (PVP) which binds water at a molecular level when it is immersed. The hydrophilic coating guarantees ultra-smooth insertion and removal, this minimises the friction more than any other traditional lubricated catheters.

NEW



HOME CATH HYDROPHILIC URINARY CATHETER WITH WATER SACHET

Homecath Hydrophilic Catheter is biocompatible. It is made of medical grade pvc tube and coated with PVP. The Smoothness dramatically reduces friction between the catheter and the urethral mucosa and minimises any risk of long term damage.

NEW




HOME CATH HYDROPHILIC URINARY CATHETER READY TO USE

Now, Homecath is available as "READY TO USE" version - a simple and safe intermittent catheter for CIC that is immediately ready for use


Homecath Ready to Use is a single use urine catheter which is made of phthalate free PVC (Polyvinyl Chloride) and coated with cross-linked PVP (Polyvinylpyrrolidone).

Homecath Ready to Use have got sterile and an activated hydrophilic surface. It is ready to be inserted into urethra without any delay. The catheter is stored in its own sterile water in aluminium pouch.

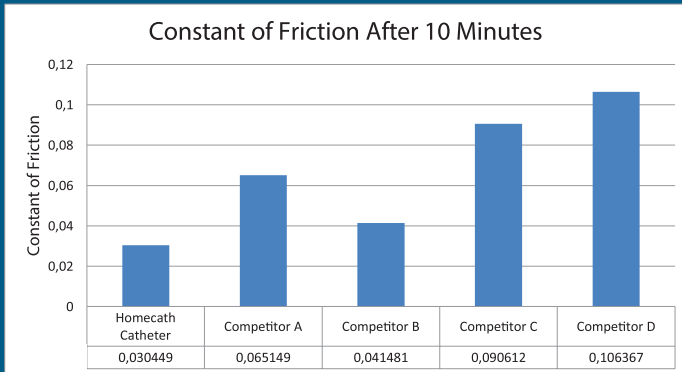


LONG LASTING DRY-OUT TIME

Homecath Catheters are inspected for dry-out time and Friction Factor Instrument tests in each batch. Test and studies show that Homecath Catheters are highly resistant against drying, the catheter will remain wet even after 10 minutes. Homecath Catheter proved its quality with its dry-out and friction factor tests.

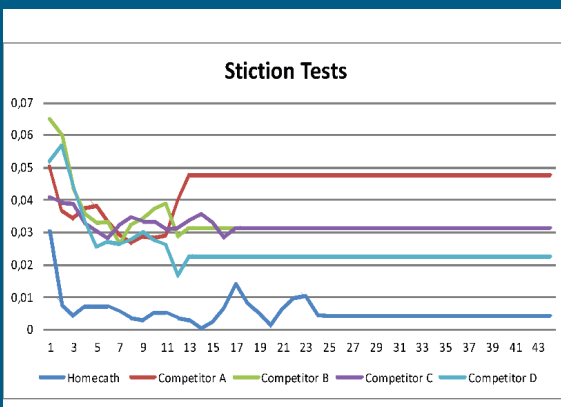

SUPERIOR SLIPPERY SURFACE

Stiction values are important values for beginning catheterization. When patient applies the catheter, first maximum friction constant is important. If it is so high, it might make stiction at urethra of the patient. Homecath Catheters have lower friction that others. It allows the Catheter to slip into urethra smoothly and reduces the pain thanks to its advanced technology.

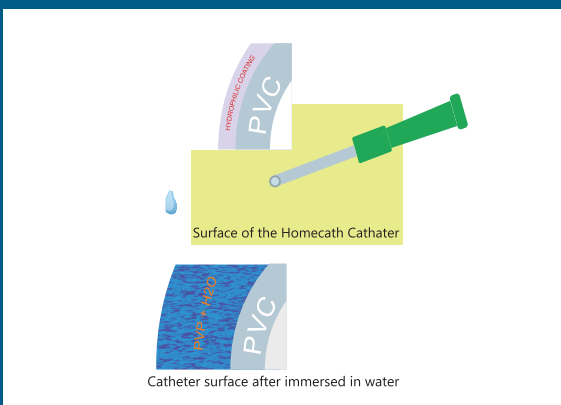

BENEFITS

After immersing Catheter into water the hydrophilic surface will absorb water inside of coated surface. This process makes catheter slippery and smooth. Unique surface of Homecath Catheter is improved for longer dry-out time and lower friction.

Homecath Catheter has been produced using only 100% phthalate-free materials.

The Catheter is extremely smooth allowing insertion and retraction is simple and pleasing to the touch.



HYDROPHILIC URINARY CATHETERS



HOME CATH HYDROPHILIC URINARY CATHETER

SIZE	MALE	FEMALE	PEDIATRIC		DILACATH	TIEMANN
CH 06	HCM 4006		HCP 2006	HCP 3006		
CH 08	HCM 4008		HCP 2008	HCP 3008		
CH 10	HCM 4010		HCP 2010	HCP 3010		HCT 4010
CH 12	HCM 4012	HCF 2012			HCD 4012	HCT 4012
CH 14	HCM 4014	HCF 2014			HCD 4014	HCT 4014
CH 16	HCM 4016	HCF 2016			HCD 4016	HCT 4016
CH 18	HCM 4018	HCF 2018			HCD 4018	HCT 4018



HOME CATH HYDROPHILIC URINARY CATHETER WITH WATER SACHET

SIZE	MALE	FEMALE	PEDIATRIC		DILACATH	TIEMANN
CH 06	HCM-SN 4006		HCP-SN 2006	HCP-SN 3006		
CH 08	HCM-SN 4008		HCP-SN 2008	HCP-SN 3008		
CH 10	HCM-SN 4010		HCP-SN 2010	HCP-SN 3010		HCT-SN 4010
CH 12	HCM-SN 4012	HCF-SN 2012			HCD-SN 4012	HCT-SN 4012
CH 14	HCM-SN 4014	HCF-SN 2014			HCD-SN 4014	HCT-SN 4014
CH 16	HCM-SN 4016	HCF-SN 2016			HCD-SN 4016	HCT-SN 4016
CH 18	HCM-SN 4018	HCF-SN 2018			HCD-SN 4018	HCT-SN 4018



HOME CATH HYDROPHILIC URINARY CATHETER - READY TO USE

SIZE	MALE	FEMALE	PEDIATRIC		DILACATH	TIEMANN
CH 06	HCM-WA 4006		HCP-WA 2006	HCP-WA 3006		
CH 08	HCM-WA 4008		HCP-WA 2008	HCP-WA 3008		
CH 10	HCM-WA 4010		HCP-WA 2010	HCP-WA 3010		HCT-WA 4010
CH 12	HCM-WA 4012	HCF-WA 2012			HCD-WA 4012	HCT-WA 4012
CH 14	HCM-WA 4014	HCF-WA 2014			HCD-WA 4014	HCT-WA 4014
CH 16	HCM-WA 4016	HCF-WA 2016			HCD-WA 4016	HCT-WA 4016
CH 18	HCM-WA 4018	HCF-WA 2018			HCD-WA 4018	HCT-WA 4018