

HYDRO TOUCH

Water Soluble Lubricating Gel

6 ml

11 ml



www.istemmedikal.com

CE 2292

LATEX FREE

STERILE R



HYDRO TOUCH

HydroTouch is a sterile water soluble lubricating gel to facilitate insertion of a tube or instrument into a body cavity. It is used for lubricating urinary catheters, endoscopes, surgical instruments etc. prior to insertion.

HydroTouch is also used in gynecological procedures for the insertion of douches, enemas, rectal thermometers and similar types of nozzles.

HydroTouch is a clear, water-soluble, non-irritating, petroleum free lubricating gel.

BENEFITS

- Syringe falls easily from protective tray enabling gold standard Aseptic Non Touch Technique (ANTT) when preparing a sterile field
- Manufacturing date, expiry date and lot number are printed on each individual syringe's tear off strip and are easily retained for accurate recording
- Syringe format is familiar to clinicians and easy to use
- Latex Free and hypoallergenic
- Eliminates the risk of suction or damage to tissue
- Easy to prime syringe for precise and controlled application
- Both 6ml and 11ml pre-filled syringes presented in a sterile package
- No risk of urethral trauma, necrosis or catheter induced urethritis.
- The risk of urinary tract infection is reduced because urethral defence mechanisms are not compromised.
- Access for cleansing and catheter changes are easier, allowing greater ability for patients to self-care.
- Allows greater comfort, particularly for patients who are chairbound.
- Allows the facilitation of trials of voiding after major surgery.

Content;

- Propylene Glycol
- Hydroxyethylcellulose
- Purified Water

Classification

Hydrotouch has been CE marked as a Class IIa Medical Device under the Medical Devices Directive 93/42/EEC.

Cost Effective Solution

Hydrotouch is a cost effective solution enabling clinicians to achieve significant savings while also ensuring best possible outcomes for the patients.






HOW TO USE

- Remove the syringe (6ml/11ml) from its sterile package by tearing off.
- Move the cap from the end of syringe.
- Instil a drop of gel to facilitate it easier.
- Insert the nozzle into the area to be lubricated and press the plunger of the syringe slowly to push out of some Lubricant Gel.

SYRINGE WITH SMOOTH TIP

The syringe enables distribution of the lubricating gel in a continuous even flow along the urethra. The smooth nozzle minimises the risk of trauma to the urethral opening.

Hydrotouch is available as sterile in disposable packages, prefilled in 6ml, 11ml in syringes.

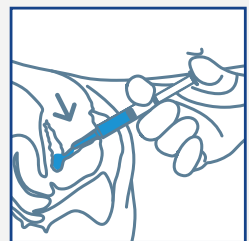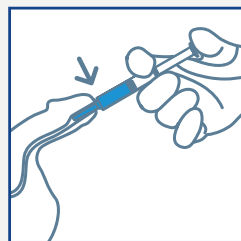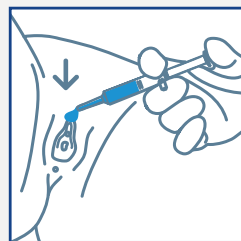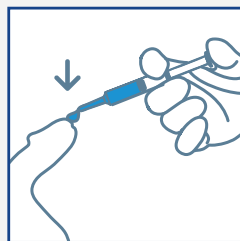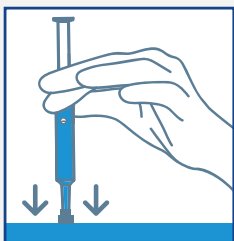

CATALOG NO.	CONTENT	QTY/ BOX
HYT 006	HYDRO TOUCH 6ml	(25x6ml/Box)
HYT 011	HYDRO TOUCH 11ml	(25x11ml/Box)