



UROLUBE[®]



6 ml



11 ml

**STERILE
LUBRICANT
GEL**
with Lidocaine

**Paraben
FREE**
CHG FREE



UROLUBE is a sterile, water soluble gel used for lubricating of the urethral catheters and other medical devices during urethral catheterization, endoscopy and cystoscopy. Beside that it can be used for rectal and colonic applications as a lubricant gel. Lubricating effect of Lubricant Gel with Lidocaine helps preventing trauma between the urethral mucosa, catheter or medical devices. The patient is relaxed and iatrogenic injures due to spasticity or unquiet are minimalized. **UROLUBE** provides optimum lubrication, while the local anaesthetic minimises pain.

COST EFFECTIVE SOLUTION

UROLUBE is a cost effective solution enabling clinicians to achieve significant savings while ensuring best possible outcomes for the patients.

UROLUBE minimises the risk of pain, infection and trauma by providing an effective lubricant that is a local anaesthetic.

BENEFITS

- Syringe falls easily from protective tray enabling gold standard Aseptic Non Touch Technique (ANTT) when preparing a sterile field
- Manufacturing date, expiry date and lot number are printed on each individual syringe's tear off strip and are easily retained for accurate recording
- Syringe format is familiar to clinicians and easy to use
- Latex Free and hypoallergenic
- Eliminates the risk of suction or damage to tissue
- Easy to prime syringe for precise and controlled application
- Both 6ml and 11ml pre-filled syringes presented in a sterile package

HOW TO USE

- Remove the syringe (6ml/11ml) from its sterile package by tearing off.
- Move the cap from the end of syringe.
- Instil a drop of gel to facilitate it easier.
- Insert the nozzle into the area to be lubricated and press the plunger of the syringe slowly to push out some Lubricant Gel with Lidocaine.

Syringe with smooth tip

The syringe enables distribution of the lubricating gel in a continuous even flow along the urethra. The smooth nozzle minimises the risk of trauma to the urethral opening.

UROLUBE is available as sterile in disposable packages, prefilled in 6ml, 11ml in syringes.






100 gram gel contains;

- Purified Water
- Propylene Glycol, Hydroxyethylcellulose
- 2g Lidocaine Hydrochloride

Classification

UROLUBE has been CE marked as a Class III Medical Device under the Medical Devices Directive 93/42/EEC.

It is classified as a Class III Medical Device in view of its primary function as a lubricating jelly, with the secondary use being the anaesthetic.

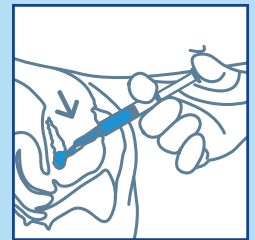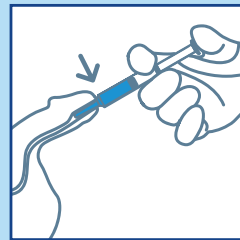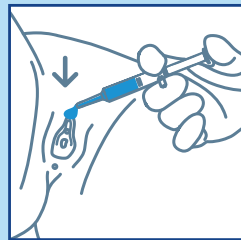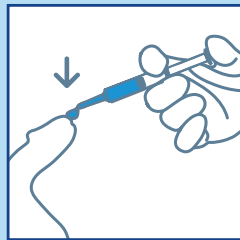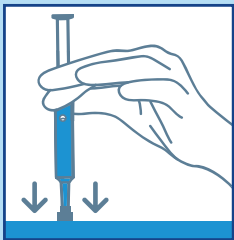

CATALOG NO.	CONTENT	QTY/ BOX
UL-L 006	UROLUBE 6ml	(25x6ml/Box)
UL-L 011	UROLUBE 11ml	(25x11ml/Box)



İSTEM MEDİKAL TIBBİ CİHAZ VE SAN. LTD. ŞTİ.
Anadolu OSB Mah. 29 Ekim Cad. No:41
Malıköy-Sincan-Ankara/TURKEY
Tel: +90 (312) 394 55 62
www.istemmedikal.com | info@istemmedikal.com